

## How it works

Pick up the **POWER-PULL** hooks and position the wheelchair in front of the ramp. Simply place the hooks on the chair and then press and hold the 'IN' button. Watch as the **POWER-PULL** takes you up the ramp. The **POWER-PULL** can also be used to exit the vehicle.

## Features

- Wireless remote for independent use
- Retractable, auto locking cable system
- Will not unravel or get tangled up
- Always pulls in any slack cable
- 'RELEASE' button for easy access

## Specifications

- Maximum capacity: 350 lbs
- 15 feet of strong, stainless steel cable

## Installation

The **POWER-PULL** takes up very little space in the vehicle and therefore is very practical. There are two applications available, one for a side entry lowered floor minivan conversion or one for a rear entry lowered floor minivan conversion.





# POWER-PULL

Ramp assist for wheelchair user



## Unique functions

If buttons are let go while going up or down the ramp, your wheelchair stays in place and you will not roll back down the ramp. The **'RELEASE'** button also has a safety feature. The cable will not release if the **POWER-PULL** is in use.

An automatic reset breaker system monitors cable tension. In the event of your wheelchair not going up the ramp straight, this system prevents you from being pulled over the side of the ramp. The **POWER-PULL** cable will hold you in place on the ramp while you reposition.

The unique clutch system disengages the motor should your wheelchair be prevented from going down the ramp. Even if the **'OUT'** button is being held, no slack is created in the cable.

*Due to the diversity of mobility solutions available to you, please contact your local mobility dealer for a personalized evaluation and overview of options tailored to your needs.*

Our products qualify as adaptive equipment and are eligible for reimbursement under the terms of Mobility programs offered by various vehicle manufacturers.

## Safety

We take our customers' safety seriously. We are dedicated to manufacturing quality mobility equipment. Our products undergo a strict certification process. Certification of our products involves different tests such as: crash testing, cycle testing and weight capacity testing. We are proud to offer a line of products that exceed the industry's standards.

Dealer :

**ADAPT SOLUTIONS**  
QUALITY MOBILITY EQUIPMENT

145 DAMASE-BRETON | SAINT-LAMBERT, QC, G0S 2W0

P 866 641 0419 | F 418 889 9838 | O 418 889 0419

[www.adaptsolutions.ca](http://www.adaptsolutions.ca)



creation : www.no1dstudio.com

[www.adaptsolutions.ca](http://www.adaptsolutions.ca)